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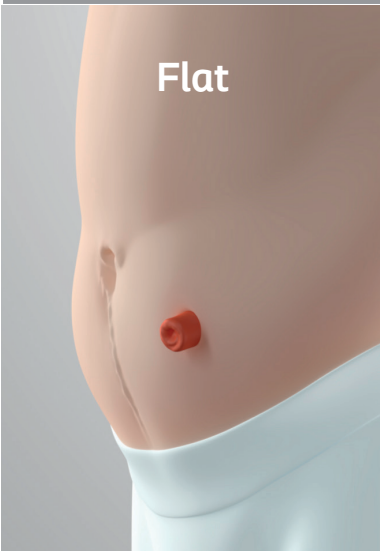
-  SenSura® Mio Convex Soft
-  SenSura® Mio Convex Light
-  SenSura® Mio Convex Deep

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Kroppsprofil	Output	Over hudnivå		I hudnivå		Under hudnivå		
		Ingen / overfladiske furer	Dype folder	Ingen / overfladiske furer	Dype folder	Ingen / overfladiske furer	Dype folder	
 Flat	Fast output	SenSura Mio (flat)	SenSura Mio (flat)	SenSura Mio Convex Soft	SenSura Mio Convex Light	SenSura Mio Convex Light (fast mage) eller Deep (myk mage)	SenSura Mio Convex Deep	
	Flytende output	SenSura Mio (flat)	SenSura Mio (flat)	SenSura Mio Convex Light	SenSura Mio Convex Light	SenSura Mio Convex Deep	SenSura Mio Convex Deep	
 Innover	Myk mage	Fast output	SenSura Mio Convex Soft	SenSura Mio Convex Light	SenSura Mio Convex Light	SenSura Mio Convex Deep	SenSura Mio Convex Deep	
		Flytende output	SenSura Mio Convex Light	SenSura Mio Convex Light	SenSura Mio Convex Deep	SenSura Mio Convex Deep	SenSura Mio Convex Deep	SenSura Mio Convex Deep
	Fast mage	Fast output	SenSura Mio Convex Soft	SenSura Mio Convex Soft	SenSura Mio Convex Light	SenSura Mio Convex Light	SenSura Mio Convex Light	SenSura Mio Convex Light
		Flytende output	SenSura Mio Convex Light	SenSura Mio Convex Light	SenSura Mio Convex Deep	SenSura Mio Convex Light	SenSura Mio Convex Light	SenSura Mio Convex Light
 Utover	Fast output	SenSura Mio Concave	SenSura Mio Concave	SenSura Mio Concave	SenSura Mio Concave	SenSura Mio Concave	SenSura Mio Concave	
	Flytende output	SenSura Mio Concave	SenSura Mio Concave	SenSura Mio Concave	SenSura Mio Concave	SenSura Mio Concave	SenSura Mio Concave	

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